

Mepilex® Border Lite

MEPILEX BORDER LITE - THE ALL-IN-ONE SOLUTION

- Minimises trauma and pain at dressing changes
- Conforms well to body contours
- Minimises the risk of maceration


SELF-ADHERENT SOFT SILICONE DRESSING

Polyurethane backing film

- Breathable
- Shower proof, bacteria and viral barrier*
*microbes larger than 25nm

Safetac® soft silicone layer

- Secure fixation
- Atraumatic to the wound and surrounding skin on removal
- Does not adhere to the moist wound bed but to dry skin only
- Reduces risk of maceration

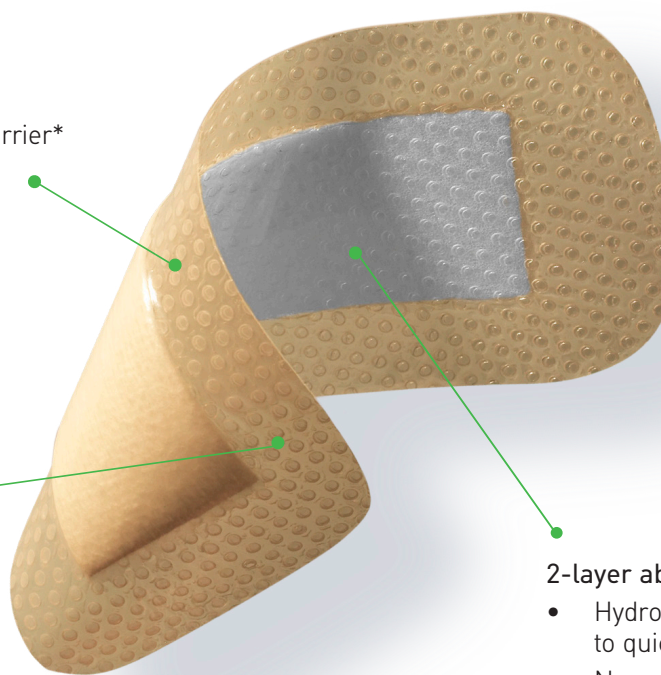

Safetac
TECHNOLOGY

2-layer absorbent foam pad


- Hydrophilic polyurethane foam to quickly absorb exudate
- Non-woven spreading layer to optimise the absorption capacity and prevent leakage.

Safetac® technology


Safetac® technology. Less trauma. Less pain.™

Safetac® is a patented soft silicone adhesive technology. Dressings with Safetac technology are atraumatic upon removal. These dressings minimise trauma to the wound and the surrounding skin, which minimises pain to the patient. The risk of maceration is minimised by sealing the wound margins.

For more information visit www.safetac.com

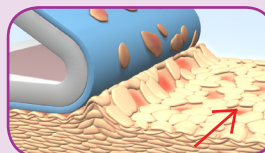


Skin stripping occurs with traditional adhesive

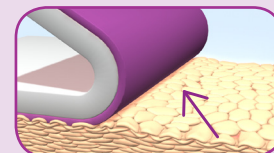


No skin stripping occurs with Safetac

How to use Mepilex Border Lite

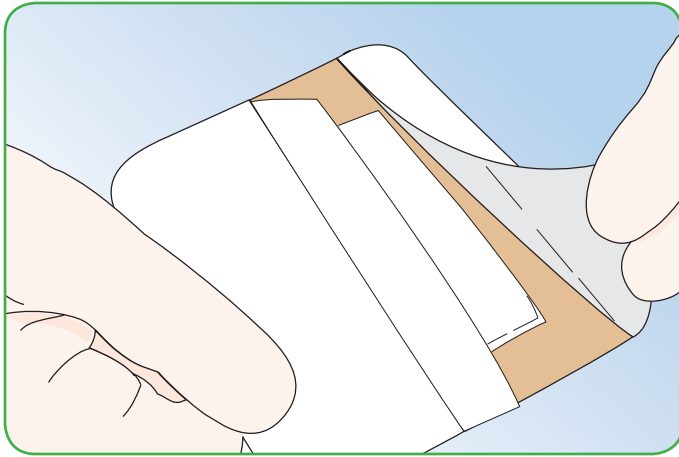

Clean the wound area. Remove the release film.

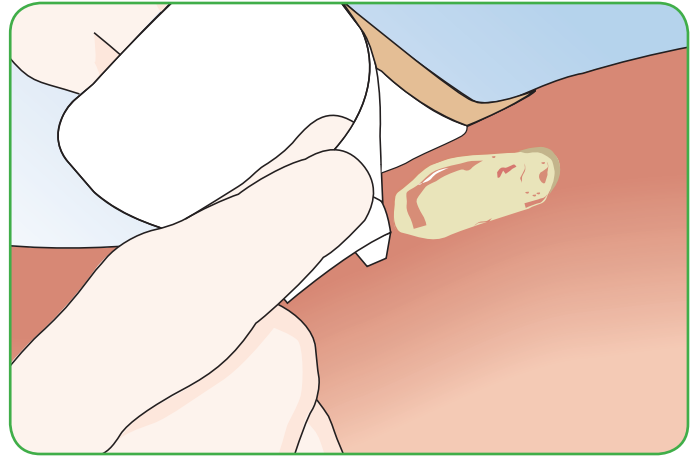

Apply the adherent side to the wound. Do not stretch.

AU144460805 Jan 2011

Benefits of Mepilex Border Lite

- Good fit to the wound due to a thin and conformable all-in-one dressing
- Broad selection of sizes
- Minimises trauma and pain during dressing changes

Note

In case of clinical signs of infection the use of Mepilex Border Lite may be continued if proper infection treatment is initiated.

Areas of use

Mepilex Border Lite is designed for the management of a wide range of non to low exuding wounds, such as leg and foot ulcers, pressure ulcers and traumatic wounds, e.g. abrasions, cuts, finger injuries, blisters and skin tears. Mepilex Border Lite can also be used as protection of compromised and/or fragile skin.



MEPILEX BORDER LITE ASSORTMENT (Sterile packed)

| Art. no | Size cm | Pcs/shelf cont. | Pcs/transp. |
|---------|-----------|-----------------|-------------|
| 281000 | 4 x 5 | 10 | 70 |
| 281100 | 5 x 12.5 | 5 | 65 |
| 281200 | 7.5 x 7.5 | 5 | 70 |
| 281300 | 10 x 10 | 5 | 50 |
| 281500 | 15 x 15 | 5 | 50 |